



SOUTH COAST  
**RADIOLOGY**

*Trusted by Doctors... Preferred by Patients*

# IntelViewer Installation

## New User

CLICK FOR  
INTELEVIEWER INSTALLERS

SCR BOOKMARK  
<https://scrconnect.com.au>

INTELEVIEWER QUICK REFERENCE  
SHORTCUT SHEET

- If you do not already have IntelViewer download onto your computer, you will need to do so. To do this you will need to log into IntelBrowser using the username and password given to you by our IT department: <https://scrconnect.com.au/IntelBrowser/app#/>
- In the main menu to the left, click 'Installers'
- Select 'IntelViewer tracks', and download the version of IntelViewer that applies  
*To check whether system 32 or 64 bit computer, go to 'Computer' or your PC, right click and select 'Properties'. This will display platform currently operating. Alternatively go to 'Control Panel', select 'System & Security', select 'System'*
- Installation wizard will launch. Read and accept terms of licence and click 'Next'
- By default, the application will be installed in: *C:\Program Files\Intelerad Medical Systems\Intelviewer*  
This can be saved in a different folder by clicking 'Choose'
- Click 'Next'
- Select 'All Users' to all automatic upgrades
- Click 'Next'
- Click 'Install'
- The IntelViewer icon will appear on your desktop. Double click to open program
- Select 'Edit Bookmark'
- Select 'Add'
- Enter South Coast Radiology server URL: <https://scrconnect.com.au>
- Select 'Protect Bookmark'
- When Logging into IntelViewer for the very first time you will be asked to click on the link to IntelBrowser to accept the terms and conditions. NOTE: The username and password will not work until these terms have been accepted